

Instruction for use
HEINE HSL 150


Instruction for use

HEINE HSL 150

DEUTSCH	4
ENGLISH	6
FRANÇAIS	8
ESPAÑOL	10
ITALIANO	12
SVENSKA	14
NEDERLANDS	16
DANSK	18
SUOMI	20
PORTUGUÊS	22

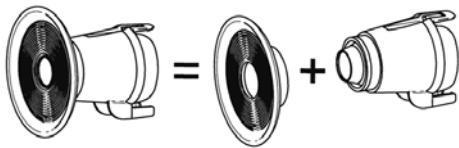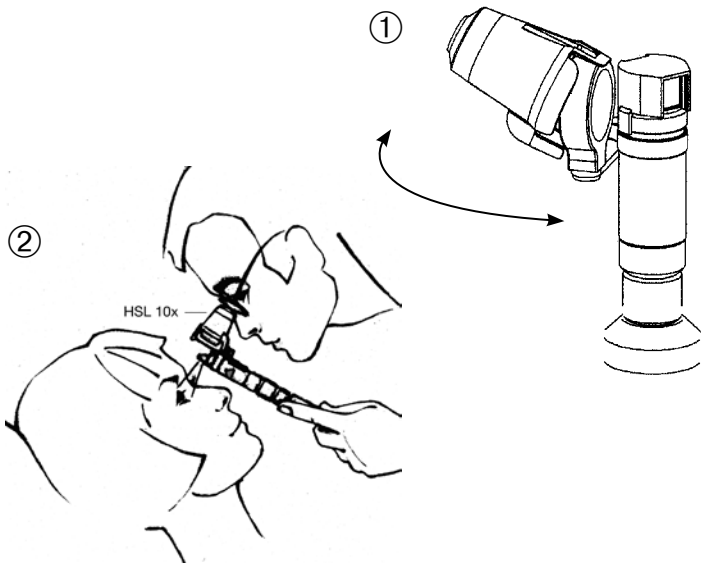③

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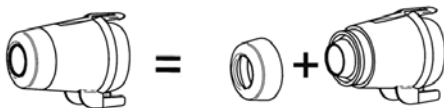 Please read and follow these instructions for use of and keep them for future reference.

Intended Use

HEINE HSL 150 is a battery powered device. The devices contain illumination and viewing optics and are intended to examine the media, cornea, aqueous, lens, vitreous and the retina of the eye. It should only be operated by qualified medical personnel within a professional healthcare facility. The instruments are intended for short-term treatment; patients can be examined according to the specified exposure guidelines.

 **For U.S. only:**
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice in terms of set up, operation or maintenance, as applicable. Notes are important, but not related to hazardous situations.

Product overview

- 1 Loupe
- 2 Filter attachment
- 3 Slide control
- 4 Connector
- 5 HEINE HSL 10x loupe attachment
- 6 Bulb
- 7 Pin
- 8 HEINE Handle

Setting up

To set up the instrument, plug the instrument head on to the HEINE handle (8).

Verify that the lamp voltage complies with the supply voltage of the bottom of the handle. The coloured marking on the bottom of the lamp shows you the lamp voltage:

White ring = HEINE XHL 2.5 V bulb
only for use with the HEINE Battery handle

Red ring = HEINE XHL 3.5 V bulb
only for use with the HEINE Rechargeable Handle

Operation

Swivel the loupe (1) to the left to examine the right eye, and vice versa. Examine the patient's right eye with your right eye and vice versa.

Approach the patient from a 30° angle to a distance of about 4cm. Looking through the loupe, sweep the eye with the slit by turning the handle.

The image can be fine-focused by adjusting the distance from the HSL to the patient's eye.

The blue filter is selected by rotating the filter attachment (2) by 180°. Pull the attachment upwards to remove.

The slide control (3) can be used to adjust the size of the slit. The loupe provides magnification in the left and right working position.

With the loupe attachment HSL 10x (5), the magnification is increased by 10x. See figures ① ② ③ (Page 3).

HEINE HSL 150 is intended for a brief examination of less than 2 minutes with a 15 minutes break until the next application.

The setup and operation of the HEINE handles are described in a separate instruction for use.

Hygienic reprocessing

The instruction is available:

- at www.heine.com
- in a paper version which you can request from the address listed

Maintenance

The device is maintenance free.

Service

The device has no components serviceable by the end-user.

General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

If you don't use the device for a longer period of time, please remove the batteries in advance.

 **Caution** – The light emitted from this instrument is potentially hazardous. The longer the duration of exposure, the greater the risk of ocular damage. Exposure to light from this instrument when operated at maximum intensity should not exceed 23 min (at a distance of 30 mm).

General Warnings

-  Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the light begins to flash.
- Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments).
 - Do not modify the device.
 - Use only original HEINE parts, spare parts, accessories and power sources.
 - Repairs shall only be carried out by qualified persons.

Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
Performance features of the ME system that have been determined to be essential to the performance	None
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the HSL 150, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	As applicable, see attachment
Test	Compliance
RF emissions CISPR11	Group 1 Class B
Harmonic Emissions	Passed*
Voltage Fluctuations/Flicker	Passed*

*n/a: "Not applicable" in the internally powered mode

Technical specification

Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Nominal voltage	3.0 V – 3.7 V
Nominal current Typ	440 – 760 mA
Protection class	internal power supply
Device classification according ISO 15004-2	Group 2

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HEINE Optotechnik GmbH & Co. KG