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ENGLISH

HEINE ophthalmic examination lamp

Please read and follow these instructions for use and keep them for future reference.

Intended Use

The HEINE ophthalmic examination light is intended to examine media, cornea, aqueous humour, lens and vitreous body of the eye. The instruments feature an optical examination system and an illumination unit powered by a battery or rechargeable battery. The instruments are intended for transient examination; patients can be examined according to the specified exposure guidelines. The product must only be used by qualified medical professionals and in professional healthcare facilities.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and Safety Information

CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! This symbol indicates valuable advice. Notes are important, but not related to hazardous situations.

Setting up

To set up the instrument, screw or fit the instrument head onto the HEINE battery handle or the HEINE rechargeable handle.

Verify that the lamp voltage complies with the supply voltage of the handle. The coloured marking on the bottom of the lamp shows you the lamp voltage:

White ring = HEINE XHL 2.5 V bulb

only for use with the HEINE Battery handle

Red ring = HEINE XHL 3.5 V bulb

only for use with the HEINE Rechargeable Handle

Operation

By rotating the ribbed control (1) you can select one of the three circular apertures, a blue filter, or a slit aperture. Use the thumb and forefinger of the hand which holds the instrument to operate the control.

The images formed by the apertures are focused at a distance of 100 mm and have the following dimensions:

Large aperture and blue filter: 10,5 mm dia.

Medium aperture: 6,5 mm dia.

Small aperture: 4,0 mm dia.

Slit aperture: 10,5 x 0,7 mm

The ophthalmic examination light is intended for a transient examination of < 2 min with a 15 minute pause before the next use.

The setup and operation of the HEINE handles are described in a separate instruction for use.

Light exposure hazard

Because prolonged intense light exposure can damage the retina, the use of the device for ocular examination should not be unnecessarily prolonged, and the brightness setting should not exceed what is needed to provide clear visualization of the target structures. The retinal exposure dose for a photochemical hazard is a product of the radiance and the exposure time. If the value of radiance were reduced in half, twice the time would be needed to reach the maximum exposure limit.

CAUTION – The light emitted from this instrument is potentially harmful. The risk of damaging your eyes increases with the exposure time. Exposure time with this instrument at maximum intensity for more than 30 sec will result in an exceeding of the guideline value for hazard.

The intensity of the light directed into the patient's eye must be reduced to a minimum level. Infants, aphakes and persons with diseased eyes will be at greater risk. The risk may also be increased if the person being examined has had any exposure with the same instrument or any other ophthalmic instrument using a visible light source during the previous 24 hours. This will apply particularly if the eye has been exposed to retinal photography.

Hygienic reprocessing

The instruction is available:

- online at www.heine.com

- in a paper version which you can request from the address listed

Maintenance

There is no regular maintenance required. The following maintenance should be done outside of the patient environment (at least 1.5 metres from the patient or patient support pursuant to IEC 60601-1) and when deemed necessary.

Changing the light source

Allow the device to cool down before changing the bulb.

Remove the ophthalmic examination light from the HEINE handle, unscrew the connector plug (2) and carefully remove the bulb.

Wipe the dome of the new bulb with a soft cloth and insert the bulb. Screw the connector plug (2) back on.

Service

The device has no components serviceable by the end-user.

General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

General Warnings

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the light begins to flash.

Do not use the device in presence of flammable gases| liquids, or in an oxygen rich environment. This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Disposal

The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests

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Electromagnetic disturbances – Requirements and tests	
<p>The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.</p>	
<p>Statement for the operational environments</p>	<p>Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.</p>
<p>Performance features of the ME system that have been determined to be essential to the performance</p>	<p>None</p>
<p>Warning</p>	<p>Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.</p> <p>Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.</p> <p>Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the ophthalmic examination lamp, including cables specified by the manufacturer. Otherwise, degradation of the perfor-mance of this equipment could result.</p>
<p>A list of all cables, transducers and other accessories that are relevant for the EMC compliance</p>	<p>See attachment</p>
Test	Compliance
<p>RF emissions CISPR11</p>	<p>Group 1 Class B</p>
<p>Harmonic Emissions</p>	<p>Passed*</p>
<p>Voltage Fluctuations Flicker</p>	<p>Passed*</p>

*n| a: „Not applicable“ in the internally powered mode

Technical specification	
<p>Environmental conditions for operation</p>	<p>+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa</p>
<p>Environmental conditions for storage</p>	<p>+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa</p>
<p>Environmental conditions for transport</p>	<p>-20 °C to +50 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa</p>
<p>Nominal voltage</p>	<p>3,0 V – 3,7 V</p>
<p>Nominal current</p>	<p>760 mA</p>
<p>Protection class</p>	<p>internal power supply</p>

XHL Xenon Halogen Bulbs	
2.5 V	X-001.88.032
3.5 V	X-002.88.047

<p>Erläuterung der verwendeten Symbole Explanation of utilized symbols Explication des symboles utilisés Explicación de los símbolos utilizados Spiegazione dei simboli utilizzati</p>	
<p>  <p>CE-Kennzeichnung kennzeichnet die Übereinstimmung mit der Europäischen Medizin-produktverordnung (EU) 2017/745. The CE mark indicates that the product complies with the medical device regulation (EU) 2017/745. Le marquage CE indique que le produit est conforme ou au règlement (UE) 2017/745 relatif aux dispositifs médicaux. La marca CE indica que el producto cumple el Reglamento europeo sobre productos sanitarios (UE) 2017/745. Il marchio CE indica la conformità con il regolamento sui dispositivi medici (UE) 2017/745.</p></p>	
<p> <p>Katalog- oder Bestellnummer Catalogue- or order number Numéro de catalogue ou de commande Número de catálogo o de pedido Codice catalogo e di ordine numero</p></p>	
<p> <p>Hersteller Manufacturer Fabricant Fabricante Produttore</p></p>	
<p> <p>Herstelldatum Date of manufacture Date de fabrication Fecha de fabricación Data di produzione</p></p>	
<p> <p>Getrennte Sammlung von Elektro- und Elektronikgeräten (Europäische WEEE Richtlinie) Product bearing this symbol may not be disposed of together with general household waste, but instead requires separate disposal according to local provisions. (European Waste Electrical and Electronic Equipment Directive, WEEE) Tri sélectif des appareils électriques et électroniques (Directive européenne DEEE) Desechado separado de aparatos eléctricos y electrónicos (Directiva Europea RAEE) Raccolta differenziata di apparecchi elettrici ed elettronici (direttiva europea RAEE)</p></p>	
<p> <p>Zulässiger Temperaturbereich in °C für Lagerung und Transport Temperature limits in °C for storage and transport Plage de température admise en °C pour le stockage et le transport Rango de temperatura permitida en °C para almacenar y transportar el producto Temperatura ammessa in °C per conservazione e trasporto</p></p>	
<p> <p>Zulässiger Temperaturbereich in °F für Lagerung und Transport Temperature limits in °F for storage and transport Plage de température admise en °F pour le stockage et le transport Rango de temperatura permitida en °F para almacenar y transportar el producto Temperatura ammessa in °F per conservazione e trasporto</p></p>	
<p> <p>Zulässige Luftfeuchtigkeit für Lagerung und Transport Humidity limitation for storage and transport Humidité admise pour le stockage et le transport Humedad del aire permitida para almacenar y transportar el producto Umidità atmosferica ammessa durante il trasporto e la conservazione</p></p>	
<p> <p>Zulässiger Luftdruck für Lagerung und Transport Pressure limitation for storage and transport Pression atmosphérique admise pendant le transport et le stockage Presión de aire permitida para almacenar y transportar el producto Pressione atmosferica ammessa durante il trasporto e la conservazione</p></p>	
<p> <p>Vorsicht Bruchgefahr! Fragile, handle with care! Fragile ! Manipuler avec soin Atención. Frágil. Attenzione: pericolo di rottura!</p></p>	
<p> <p>Trocken lagern! Keep dry! Conservar au sec ! Conservar en un lugar seco! Evitare ambienti umidi!</p></p>	
<p> <p>Gebrauchsanweisung verbindlich befolgen. (Hintergrundfarbe: blau, Vordergrundfarbe: weiß) Follow instructions for use! (Background color: blue, foreground color: white.) Suivre le mode d’emploi. (Couleur de fond : bleu ; couleur du premier plan : blanc) Seguir obligatoriamente las instrucciones de uso. (Color de fondo: azul, color de primer plano: blanco) Attenersi obbligatoriamente alle istruzioni per l’uso. (Colore dello sfondo: blu, colore in primo piano: bianco)</p></p>	
<p> <p>Unique Device Identification</p></p>	
<p> <p>Medical Device</p></p>	

<p>med 6996</p>	<p>2022-03-29</p>
<p> HEINE Optotechnik GmbH & Co. KG Dornierstr. 6, 82205 Gilching, Germany www.heine.com</p>	