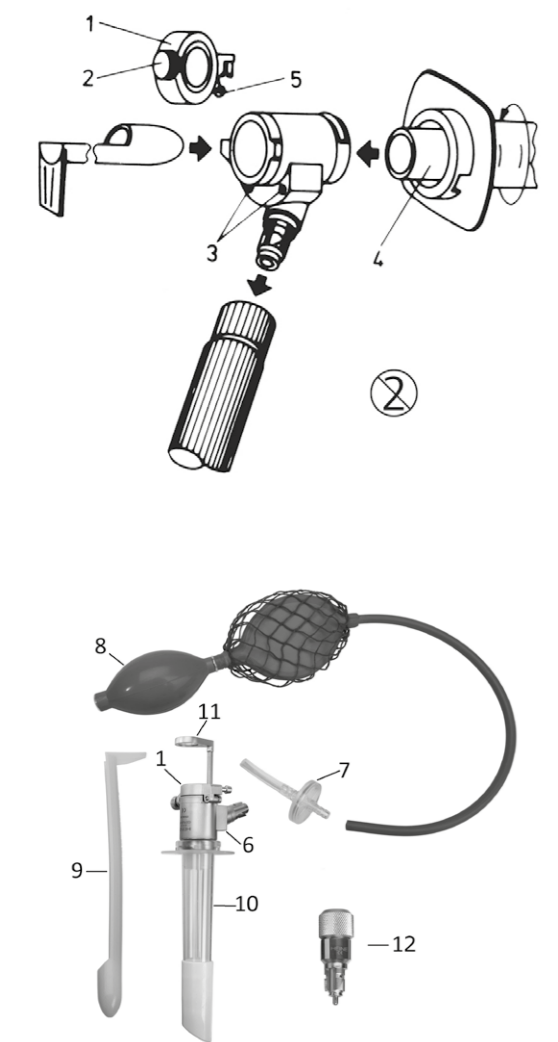
Instruction for use HEINE UniSpec® Instrument Head

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DEUTSCH

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HEINE UniSpec Instrument Head

ENGLISH

Please read and follow these instructions for use and keep them for future reference.

Intended Use
The HEINE UniSpec Instrument Head and the UniSpec disposable tubes with fibreglass illumination are intended for examination and performance of procedure within the rectum and the anus. The product may only be used by qualified medical professionals and in professional healthcare facilities.

For U.S. only:
Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and Safety Information
 CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to serious extent. (Background color yellow; foreground color black).
 NOTE! This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

Product overview (see figure)

1 Viewing window	5 Connector for the Hygiene Filter	9 Obturator
2 Viewing screw	6 Instrument Head for UniSpec Tubes	10 Tube with extension-piece
3 Hole for the optical magnification	7 Hygiene Filter	11 Swivel lens
4 Extension-piece	8 Double bulb insufflator	12 Illumination adapter

Setting up
Loosen the fastening screw (2) and open the viewing window (1). If the viewing window isn't necessary, it can be removed by extracting it upwards out of the instrument head.
Attach the instrument to a suitable HEINE power/light source. Setup and operation of the HEINE power/light sources are described in separate instructions for use.
 When using a HEINE BETA handle, attach the HEINE illumination adapter (12) between the instrument head and the handle.

Pull the obturator out of the tube. Insert the tube with the extension-piece (4) as far as possible into the instrument head and secure this by twisting to the right. Insert the obturator (9) through the instrument head into the tube (10).

Apply a suitable lubricant to the tip of the obturator and the tube.
 The tubes should only be inserted into the patient when an obturator is fitted.
Choose a tube size suitable for the patient.

The instrument head can be used with the rubber insufflation bulb (8). To do so, connect the rubber insufflation bulb with the hygiene filter (7) to the viewing window (5). Control the connection. If it isn't securely attached, fix it more tightly.
 The use of the instrument without the HEINE hygiene filter may lead to contamination of the insufflation bulb and thereby to an infection of the following patients. Therefore always use the filter. If you have forgotten to use the filter, dispose the insufflation bulb.
If there is a risk of contamination inside the rubber insufflation bulb, the insufflation bulb has to be disposed.
The hygiene filter is a single use product. If it is reused, the risk of cross contamination to the following patient increases. The hygiene filter is not suitable for cleaning, disinfection or sterilization.

Risk of allergic shock: The *orange rubber insufflation bulb* contains natural rubber latex. Do not use it in patients with a latex allergy.
 The *black insufflation bulb* is latex- and phthalate free.

Operation
When the tube is inserted, remove the obturator through the instrument head.
The viewing window is equipped with a face plate and can be conveniently opened and closed at any time. The instrument can be made airtight by slightly tightening the fastening screw (2). For an insufflation bulb without a valve, the insufflated air can be regulated by gently loosening or tightening the screw.
If magnification of the object is required, insert the swivel lens into one of the two holes provided (3). The loupes are marked "P" for procto- and anoscope and "R" for recto- or sigmoidoscope.

Application duration
The sigmoidoscopes, proctoscopes and anoscopes are intended for transient application. The application time should not exceed 15 minutes.

Hygienic reprocessing
The instruction is available:
- at www.heine.com
- in a paper version which you can request from the address listed

Maintenance
The device is maintenance free.

Service
The device has no components serviceable by the end-user.

General Notes
 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.
The expected life cycle, when the UniSpec instrument head is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.
The UniSpec disposable tubes can be used up to 5 years from the date of manufacture.
Note to the user and/or patient:
All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.
Only disconnect the instrument head from the power supply if the device is turned off.

General Warnings
 Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.
Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments).
Do not modify the device.
Use only original HEINE parts, spare parts, accessories and power sources.
The device must not be placed near strong Magnetic fields, e.g. MRI units.
Check the device for contaminates before use. If necessary, clean the device or dispose of it in the event of non-removable contamination.
In order not to exceed the surface temperature of 41 °C of the application part, keep within the application duration.

Disposal
 The product must be recycled as separated electrical and electronic devices.
 Please observe the relevant state-specific disposal regulations.

The appendix contains following tables
- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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- Distúrbios eletromagnéticos – Requisitos e testes

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- Esclarecimento sobre os símbolos utilizados

Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
Performance features of the ME system that have been determined to be essential to the performance	None

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the UniSpec instrument head, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

A list of all cables, transducers and other accessories that are relevant for the EMC compliance	As applicable, see attachment0
Test	Compliance
RF emissions CISPR11	Group 1 Class B
Harmonic Emissions	Passed*
Voltage Fluctuations/Flicker	Passed*

*n/a: „Not applicable“ in the internally powered mode

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Technical specifications	
Environmental conditions for operation	+10 °C to +35 °C <p>10 % to 75 % rel. humidity</p> <p>700 hPa to 1060 hPa</p>
Environmental conditions for storage	+5 °C to +45 °C <p>45 % to 80 % rel. humidity</p> <p>500 hPa to 1060 hPa</p>
Environmental conditions for transport	-20 °C to +50 °C <p>45 % to 80 % rel. humidity</p> <p>500 hPa to 1060 hPa</p>
Protection class	IP 20

Suitable power/light source	
HEINE BETA Handle 3,5 V and the Illumination Adapter	X-002.98.002
BETA 4 NT rechargeable handle	X-007.99.396
BETA 4 USB Li-ion rechargeable handle	X-007.99.387
BETA 4 SLIM NT rechargeable handle	X-007.99.474
BETA L Li-ion rechargeable handle	X-007.99.395
BETA NT rechargeable handle	X-002.99.411
BETA SLIM rechargeable handle	X-002.99.471
Lamp Handle with switch and XHL Xenon Halogen bulb	X-004.99.607
Sigmoidoscope / Proctoscope KIT RE7000	E-096.15.501

Accessories	
HEINE UniSpec Disposable Anoscopes <p>25 pcs. / 85x20mm</p> <p>100 pcs. / 85x20mm</p>	E-003.19.925 <p>E-003.19.911</p>
HEINE UniSpec Disposable Proctoscopes <p>25 pcs. / 130x20mm</p> <p>100 pcs. / 130x20mm</p>	E-003.19.825 <p>E-003.19.811</p>
HEINE UniSpec Disposable Sigmoidoscopes <p>25 pcs. / 250x20m</p> <p>100 pcs. / 250x20mm</p>	E-003.18.825 <p>E-003.18.811</p>
Rubber insufflation bulb (orange)	E-003.18.105
Insufflation bulb (black)	E-003.18.105
Hygiene Filter (25 pcs.)	E-000.18.116
HEINE Illumination adaptor	X-002.98.002

Options	
Swivel lens (only suitable for sigmoidoscopes)	E-003.18.096
Swivel lens (only suitable for proctoscopes and anoscopes)	E-003.19.099
Viewing window	E-003.18.099

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