Instruction for use
HEINE mini-c
HEINE mini-e

Instruction for use

HEINE mini-c

HEINE mini-e

DEUTSCH	3
ENGLISH	6
FRANÇAIS	9
ESPAÑOL	12
ITALIANO	15
SVENSKA	18
NEDERLANDS	21
DANSK	24
NORSK	27
SUOMI	30
PORTUGUÊS	33

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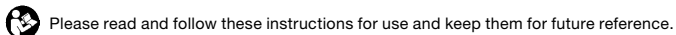
Intended Use

The hand-held, internally powered HEINE mini-c is exclusively intended for non-contact and transient examination of the skin, anterior segment of the eye and the pupil reflex.

The hand-held, internally powered HEINE mini-e is intended exclusively for illuminating the transition area from the auricle to the external auditory canal.

The device may only be used by qualified medical personnel and in professional healthcare facilities.

 **For U.S. only:**
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

 **CAUTION!** This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

Product overview

- 1 Handle
- 2 On | Off Switch
- 3 Head piece
- 4 Clip

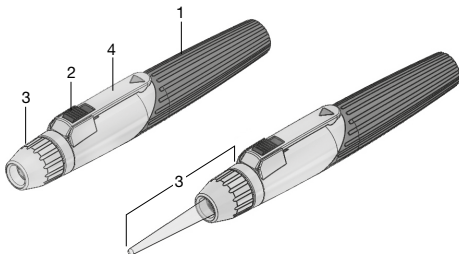
Fig. 1 mini-c

mini-e

Setting up

To set up, unscrew the head piece (3) from the handle (1) and insert two alkaline primary cells (size LR03 | AAA | 1.5V) (Fig. 3). Then screw the head piece back on.

Operation

mini-c and mini-e

To turn the instrument on, slide the switch down. When the device is switched on, a green marker is visible. To turn the device off, slide the switch back up.

When the clip is clipped onto a pocket the handle turns off automatically.

mini-c

During examination of the pupil reflex, a distance of about 50 mm is required. If a working distance greater than 100 mm is chosen, the exposure to light decreases.

Hygienic reprocessing

The instruction is available:

- at www.heine.com

- in a paper version which you can request from the address listed

Maintenance

Changing the batteries

- ⚠ Change the batteries outside of the patient environment (at least 1.5 metres from the patient or patient support pursuant to IEC 60601-1, see Fig. 2). Please let the device cool down before changing the battery.

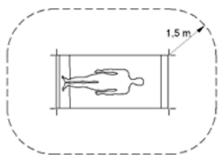
Fig. 2

Unscrew the head piece (3) and shake out the batteries. Insert new batteries with the plus contact facing the handle head (Fig. 3). Then screw the head piece back on.

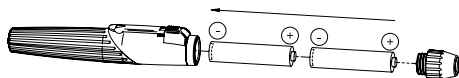
Fig. 3

Service

The device has no components serviceable by the end-user.
The LED cannot be changed on this instrument.

General Notes

- ⚠ The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and | or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

Store and use the device only in a dry and dust-free environment.

Let the device accommodate to ambient conditions when it is cold.

General Warnings

- ⚠ Check the correct operation of the device before use! Do not use the device if there is a failure of the illumination, the switch isn't moveable or if there are visible signs of damage. Check that the lamp lights up after inserting the batteries. If there is failure of the illumination, turn the device off and check if the batteries have been inserted correctly. Do not use the device in presence of flammable gases | liquids, or in an oxygen rich environment. Do only use the batteries specified by HEINE. In order to ensure a secure electrical connection, use only batteries from renowned manufacturers. Use only original HEINE parts, spare parts, accessories and power sources. This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners.

Repairs shall only be carried out by qualified persons.

Do not modify the device.

Do not insert empty cells into the device.

If the device is stored for a longer period of time without being used, please remove the batteries.

Avoid the penetration of humidity.

Do only use alkaline primary cells!

If the device gets warm, switch off the device, let the device cool down and remove the batteries.

Light exposure hazard (mini-c)

 Because prolonged intense light exposure can damage the retina, the use of the device for ocular examination should not be unnecessarily prolonged, and the brightness setting should not exceed what is needed to provide clear visualization of the target structures. The retinal exposure dose for a photochemical hazard is a product of the radiance and the exposure time. If the value of radiance were reduced in half, twice the time would be needed to reach the maximum exposure limit.

While no acute optical radiation hazards have been identified for the HEINE mini-c Examination Light, it is recommended that the intensity of light directed into the patient's eye be limited to the minimum level which is necessary for diagnosis. Infants, aphakes and persons with diseased eyes will be at greater risk. The risk may also be increased if the person being examined has had any exposure with the same instrument or any other ophthalmic instrument using a visible light source during the previous 24 hours. This will apply particularly if the eye has been exposed to retinal photography.

ISO 15004-2: Group 2 (LED)

Caution - The light emitted from this instrument is potentially hazardous. The longer the duration of exposure, the greater is the risk of ocular damage. An exposure time with this instrument at maximum intensity of longer than (see table exposure guideline) will lead to the guideline hazard value being exceeded.

ANSI Z80.36-2016: Group 2 (LED)

Caution - The light emitted from this instrument is potentially hazardous. The longer the duration of exposure, the greater is the risk of ocular damage. Exposure to light from this instrument when operated at maximum intensity will exceed the recommended maximum exposure (RME) of 2.2 J/cm², unless additional action is taken by the user to minimize exposure, after __ min (see table exposure guideline). The risk of retinal injury at an exposure of 2.2 J/cm² is not high, but because some patients may be more susceptible than others, caution is advised if this radiant exposure value is exceeded. However, because of a significant risk of injury at exposures exceeding 10 J/cm², the user should avoid exposures longer than __ min. (see table exposure guideline).

Disposal

-  The product must be recycled as separated electrical and electronic devices.
-  Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Exposure guidelines
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	<p>Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.</p> <p>The supply voltage quality should be that of a typical hospital environment.</p> <p>Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.</p>
Performance features of the ME system that have been determined to be essential to the performance	None
Necessary instructions for maintaining basic safety and essential performance with regards to electromagnetic disturbances for the expected life cycle	<p>Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.</p> <p>Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.</p> <p>Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the mini-c and mini-e, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.</p>
Warning	<p>Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.</p> <p>Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.</p> <p>Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the mini-c and mini-e, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.</p>
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	EMC compatibility is only ensured if original HEINE spare parts, accessories and power sources are used. The EMC compatibility when using power sources from other manufacturers must be evaluated by the user.
Test	Compliance
RF emissions CISPR11	Group 1, Class B
RF emissions (EN 55011 CISPR 11)	Passed
Conducted emissions (EN 55011 CISPR 11)	Not applicable, because of internally powered mode
Radiated emissions (EN 55011 CISPR 11)	Passed
Harmonic current emissions (IEC 61000-3-2)	Not applicable, because of internally powered mode
Voltage changes, voltage fluctuations and flicker (IEC 61000-3-3)	Not applicable, because of internally powered mode
Immunity	See attached immunity test levels

Immunity test levels

Test	Test level	
	IEC 60601-1-2 test levels	Compliance test levels
Electrostatic Discharge (IEC 61000-4-2)	Contact Discharge: ± 8 kV Air Discharge: ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV	
Radiated RF EM fields (IEC 61000-4-3)	3 V/m 80–2700 MHz 80 % AM at 1 kHz	
Electrical fast transients bursts (IEC 61000-4-4)	± 2 kV 100 kHz repetition frequency	Not applicable, because of internally powered mode
Surges (IEC 61000-4-5)	± 0.5 kV, ± 1 kV	
Conducted disturbances induced by RF fields (IEC 61000-4-6)	3 V 0.15 MHz – 80 MHz 6 V in ISM bands between 0.15 MHz and 80 MHz 80 % AM at 1 kHz	
Proximity fields from RF wireless communications equipment (IEC 61000-4-3)	385 MHz; Pulse Modulation: 18 Hz; 27 V/m 450 MHz, FM: ± 5 Hz deviation: 1 kHz sine; 28 V/m 710, 745, 780 MHz; Pulse Modulation: 217 Hz; 9 V/m 810, 870, 930 MHz; Pulse Modulation: 18 Hz; 28 V/m 1720, 1845, 1970 MHz; Pulse Modulation: 217 Hz; 28 V/m 2450 MHz; Pulse Modulation: 217 Hz; 28 V/m; 5240, 5500, 5785 MHz; Pulse Modulation: 217 Hz; 9 V/m	
Power frequency magnetic fields (IEC 61000-4-8)	30 A/m; 60 Hz	
Voltage dips (IEC 61000-4-11)	0 % U_T ; 0.5 cycle; at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % U_T ; 1 cycle and 70 % U_T ; 25/30 cycles Single phase: at 0°	Not applicable, because of internally powered mode
Short interruptions (IEC 61000-4-11)	0 % U_T ; 250/300 cycles	

Exposure Guidelines (mini-c)

A working distance of 100 mm results in the highest light exposure.

If the distance is smaller or larger than 100 mm, the light exposure is reduced.

Working distance	Duration according to ANSI Z80.36-2016	Duration according to EN ISO 15004-2:2007
100 mm	Recommended maximum: 58 seconds	5 minutes and 40 seconds

Technical specification

mini-c classification according to ISO 15004-2 and ANSI Z80.36	In 100 mm distance group 2	
Weight	<u>mini-c</u> 45 g incl. batteries	<u>mini-e</u> 45,5 g incl. batteries
Power supply	2 x 1.5 V alkaline battery (size AAA/LR03)	
Applied part	Ear light Type BF	

Environmental conditions

Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa

Erläuterung der verwendeten Symbole
Explanation of utilized symbols
Explication des symboles utilisés
Explicación de los símbolos utilizados
Spiegazione dei simboli utilizzati
Förklaring av symboler som används
Verklaring van de gebruikte symbolen
Forklaring af de anvendte symboler
Symbolforklaring
Käytettyjen symbolien selitys
Explicação dos símbolos utilizados

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Product bearing this symbol may not be disposed of together with general household waste, but instead requires separate disposal according to local provisions.

(European Waste Electrical and Electronic Equipment Directive, WEEE)

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