



Reprocessing Agents for HEINE visionPRO

HEINE Optotechnik GmbH & Co. KG has tested and approved the following agents for material compatibility for disinfection of the HEINE visionPRO video laryngoscope.

Manufacturer	Product name
B.Braun	Meliseptol Wipes Sensitive
	Meliseptol Wipes Ultra
Bode Chemie / Hartmann	Bacillol Tissues
	Mikrobac Tissues
	Mikrobac viruzidal
DIALEX BIOMEDICA	clinell universal wipes
Dr. Schuhmacher	Cleanisept Wipes
	Cleanisept Wipes forte
Ecolab Healthcare	Incidin Alcohol Wipe
	Sani-Cloth Active
Johnson & Johnson MEDICAL	Cidex OPA
Metrex	CaviWipes
PDI	Sani-cloth AF3
	Super Sani-cloth
Schülke	Mikrozid AF wipes
	Mikrozid universal wipes
Tristel	Tristel Trio Wipes*
Osaki Medical	One step wipes
SARAYA	Surface Sanitizing wipes

The validated reprocessing procedure is described in the reprocessing instruction MED235566.

* Note that the use of disinfectants may cause colour changes on the surface of this medical device. However, such changes do not result in performance limitations and there is no risk to patients, users and third parties.



The following disinfectant active ingredients **must NOT** be used for disinfection of the HEINE visionPRO video laryngoscope. This list is not exhaustive. There may be other active ingredients that are not compatible.

disinfectant active ingredients
Hydrogen peroxide
Peracetic acid