

Hygienic reprocessing

HEINE BETA® 200 Retinoscopes with HEINE ParaStop®

General warning and safety information



WARNUNG! This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent.



HINWEIS! This symbol indicates valuable advice. Notes are important, but not related to hazardous situations.

\$	Instructions on hygienic reprocessing must be adhered to, based on national standards,
	laws and guidelines.
	The described reprocessing measures do not replace the specific rules applicable for your
	institution/ department.
<u> </u>	In the event of suspected contamination, carry out hygienic reprocessing of the
	instrument.
	HEINE Optotechnik GmbH & Co. KG only approves the agents and procedures listed in
	this instruction.
	Hygienic reprocessing is to be carried out by persons with adequate hygienic expertise.
	Please consider the instructions of the manufacturer for the applied reprocessing media.
	Avoid liquid entry into the device.
Limitations on	Periodically check the integrity of the device.
reprocessing	



MED 113939 03/06.2020



Wipe disinfection

1. **Preparation**

Switch off the device and let it cool down.

Manual cleaning and disinfection



Pay attention that all surfaces are completely moistened for the complete exposure time specified by the disinfectant manufacturer. If necessary, increase the number of wiping procedures and or the number of wipes.

Equipment

- Cleaning agent, if necessary: enzymatic (e.g. neodisher MediClean)
- Disinfectant:

alcoholic (e. g. Incides N)

alkoholic + quarternary ammonium compounds (e. g. Super Sani-Cloth)

If you use other than the recommended reprocessing agents, make sure that they do not foam.

Implementation

- For heavier soiling, you can first clean with a wipe soaked with cleaning agent before disinfecting with a disinfectant wipe.
- Clean and disinfect the device manually.
- Pay particular attention to difficult to access areas.
- Thoroughly wipe the touched areas: e. g. operating elements, rotary controls, locking elements.
- For removing the disinfectant and drying afterwards, follow the instructions provided by the disinfectant manufacturer.

Inspection and function testing



Check the device for any visible contaminants or abrasions. Reprocess again if necessary. Dispose if the contaminants cannot be removed.

Storage

Store it in such a way that it is protected from recontamination, dust and moisture.



MED 113939 03/06.2020