

Hygienic reprocessing

HEINE Examination Lights EL3 LED and EL 10 LED®

General warning and safety information



WARNING! This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent.



NOTE! This symbol indicates valuable advice. Notes are important, but not related to hazardous situations.

	Instructions on hygienic reprocessing must be adhered to, based on national
	standards, laws and guidelines.
	The described reprocessing measures do not replace the specific rules applicable for
	your institution/ department.
<u>^</u>	In the event of suspected contamination, carry out hygienic reprocessing of the
	instrument.
	HEINE Optotechnik GmbH & Co. KG only approves the agents and procedures listed in
	this instruction.
	Hygienic reprocessing is to be carried out by persons with adequate hygienic expertise.
	Please consider the instructions of the manufacturer for the applied reprocessing
	media.
	Avoid liquid entry into the device.
	Before reprocessing, disconnect the device from the power source.
	The plug-in power supply must not be reprocessed.
€	Before using it again, ensure that the device is completely dry after reprocessing.
Limitations on	Periodically check the integrity of the device.
reprocessing	



MED 234480 01/04.2021



Wipe disinfection

1. Preparation

Switch off the device and let it cool down.

2. Manual cleaning and disinfection



Pay attention that all contaminated surfaces are completely moistened for the complete exposure time specified by the disinfectant manufacturer. If necessary, increase the number of wiping procedures and/or the number of wipes.

Equipment

Disinfectant:

alcoholic (e. g. Incides N or mikrozid universal wipes), quarternary ammonium compounds (e. g. Cleanisept Wipes , Mikrobac Tissues or Sani-Cloth AF3) or

alcoholic + quarternary ammonium compounds (e. g. Super Sani-Cloth)

Implementation

- Clean and disinfect the contaminated surfaces manually by wiping.
- Thoroughly wipe the touched areas.
- For removing the disinfectant and drying afterwards, follow the instructions provided by the disinfectant manufacturer.
- Residues on optics may be removed using a low-bacterial, dry, micro-fibre cloth.
- Clean the cable and the power supply unit only if necessary and only with a dry cloth.

3. Inspection and function testing



• Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

4. Storage

Store it in such a way that it is protected from recontamination, dust and moisture.



MED 234480 01/04.2021