


∴ HEINE OMEGA 500 Breath Shield


Breath Shield for a clear view during examinations with the HEINE OMEGA 500 Binocular Indirect Ophthalmoscope.

∴ **Optimal view** through the ophthalmoscope thanks to a cut-out on the optics.

∴ **Easy to attach.**

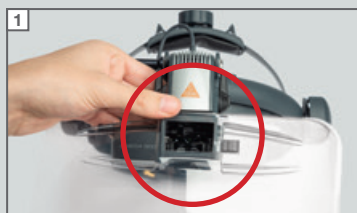

Place the Breath Shield on the front of the instrument under the HEINE logo, so that the cut-out in the Breath Shield is positioned directly over the optical window.

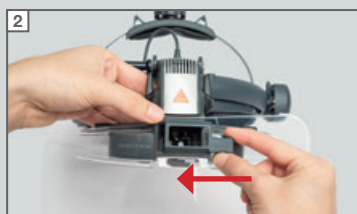

Take the mounting clip supplied and push it onto the guide tab on the instrument.

∴ **Cleaning and disinfection:**



For cleaning and disinfection, remove both the transparent Breath Shield and the mounting clip from the instrument.

Inspect the Breath Shield for damage or contamination before each use. If damage or irremovable contamination is identified, stop using the Breath Shield and dispose of it. Disinfect the product before disposal. Dispose of the Breath Shield in accordance with local regulations.

Detailed instructions for cleaning and disinfecting the Breath Shield can be found at [heine.com/en_us/breath-shield](https://www.heine.com/en_us/breath-shield)

∴ Lightweight, rigid material.

∴ Dimensions (width x length): 231 mm x 205.08 mm.

∴ For use with the HEINE OMEGA 500 only.

Ordering options

You can order Breath Shields from your medical equipment supplier. Available individually or as a set of three.

Article numbers

- ∴ HEINE OMEGA 500 Breath Shield (1 x Breath Shield, 1 x mounting clip): **C-000.33.016**
- ∴ HEINE OMEGA 500 Breath Shield (3x Breath Shield, 1 x mounting clip): **C-000.33.017**