

# HEINE® Cube Quick Start Guide

1. Connect your HEINE Cube to the mains power supply via the supplied power cable. Press the ON button to start the HEINE Cube.



2. Connect your device to the HEINE Cube wireless network.



SSID: HEINE-Cube  
Password: heinecube

3a. With a  
Mac / Windows / Tablet



open one of the  
standard web browsers



open the HEINE Cube address  
192.168.88.1

Login

3b. With an iPhone\* version 5s  
and upwards



download the HEINE Cube App



open the App and type in the HEINE  
Cube address 192.168.88.1

Login

LOGIN

Please login to unlock the application

User

Password

Login

